

# ALTERNATE AQUACULTURE VACCINE DELIVERY SYSTEM

TEAM FISH VAC BEN, CODY, HASAN, KHARI




# PROBLEM BACKGROUND

Preserving health in animal kingdoms requires the assistance of a diverse group of specialists. With the innovation in technology, we can use this to combat diseases in natural habitats. As a team of engineers, it is our jobs to design and construct instruments that improves the ease of field work. In our case, assisting the College of Natural Resources.







# INITIAL CLIENT INTERVIEW

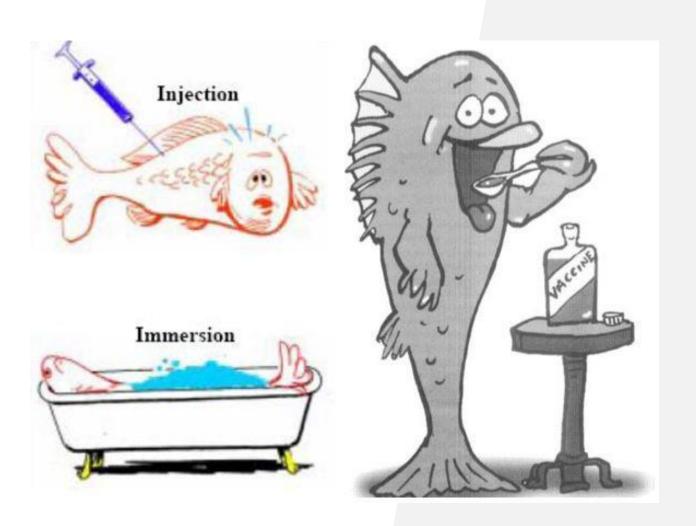



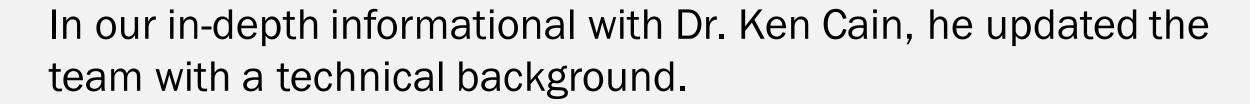

- 3 ways to vaccinate fish: Injection, Oral, and Immersion.
- The immersion technique in used when treating large schools of fish
- When using this method, the immersion time and effectiveness correlate due to the skin-vaccine contact. It takes time for the skin to absorb the treatment.
- The most ideal way to shorten the immersion time without decreasing the effectiveness, would be to perforate the skin of the fish harmlessly, and continue with the process

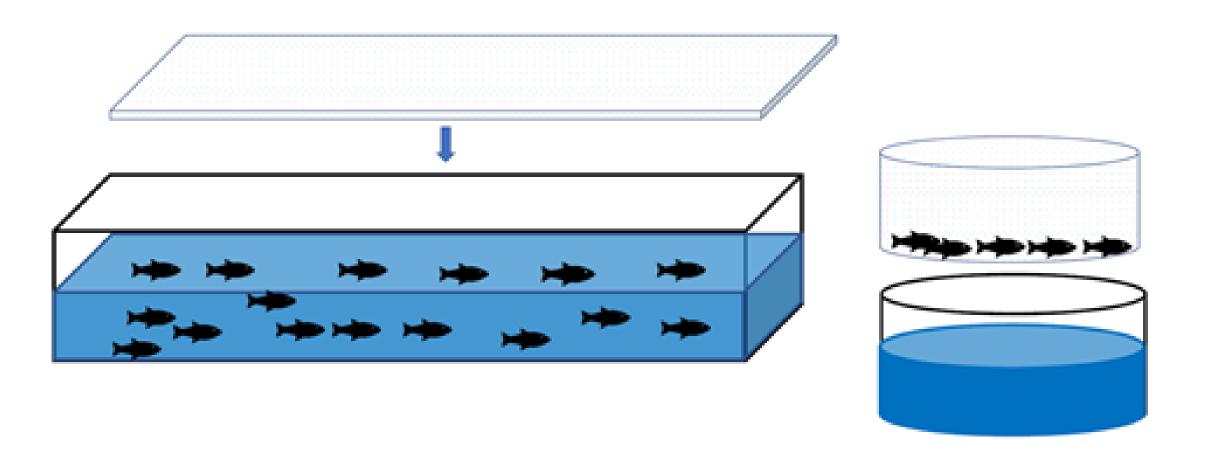



# PROJECT REQUIREMENTS

- Should be used with existing vaccination tanks
- Should contain non-specific parts where possible to allow for easy repair
- Two Prototypes (Lab model and production model)
- Should be able to be operated and moved by the average person








# INITIAL DESIGN CONCEPT

## BED OF NEEDLES:

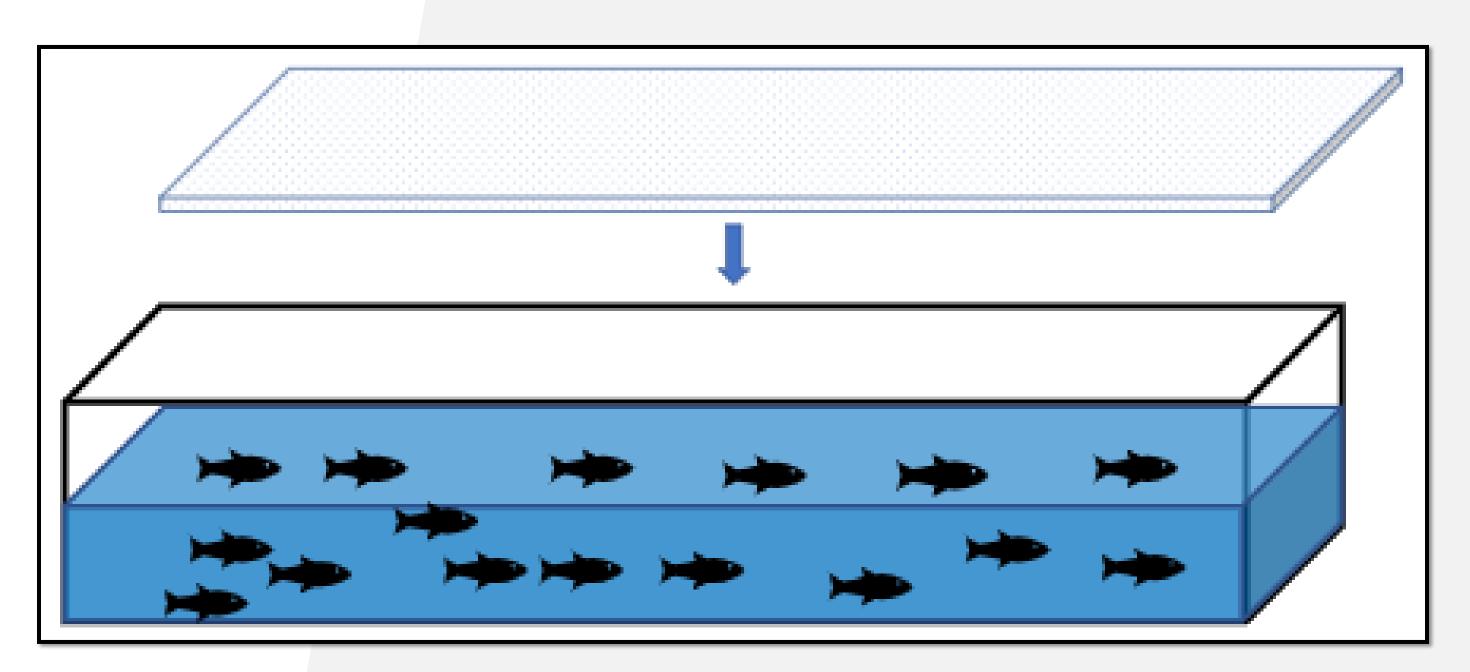

- Requires water to be drained from the tank or a large platform to be lowered into tank
- Would require 1000's of needles
- A bed of needles isn't a readily available item



# MICRONEEDLE ROLLERS



After coming up empty handed with Microneedle sheets, we ran into a common and popular skin treatment method: Derma-Rolling or Micro-Needling.

This product is a small cylindrical roller with exposed microneedles. This product is commonly used to pierce the skin of humans, to agitate, and promote collagen in areas.

#### Advantages of this Product

- Adjustable
- Easily Replaceable
- Applicable
- Cheap

# VALUE PROPOSITION




- Vaccinating large quantities of fish efficiently saves money and strenuous effort
- It is our responsibility to support our ecosystems and resources
- Creating an instrument that is easy to use and cheap to maintain is important for the consumer and the supplier.
- An effective and well-engineered product is our goal.



# DESIGN IDEAS IMPLEMENTING USE OF MICRONEEDLE ROLLERS

## CHUTE AND RAMP:

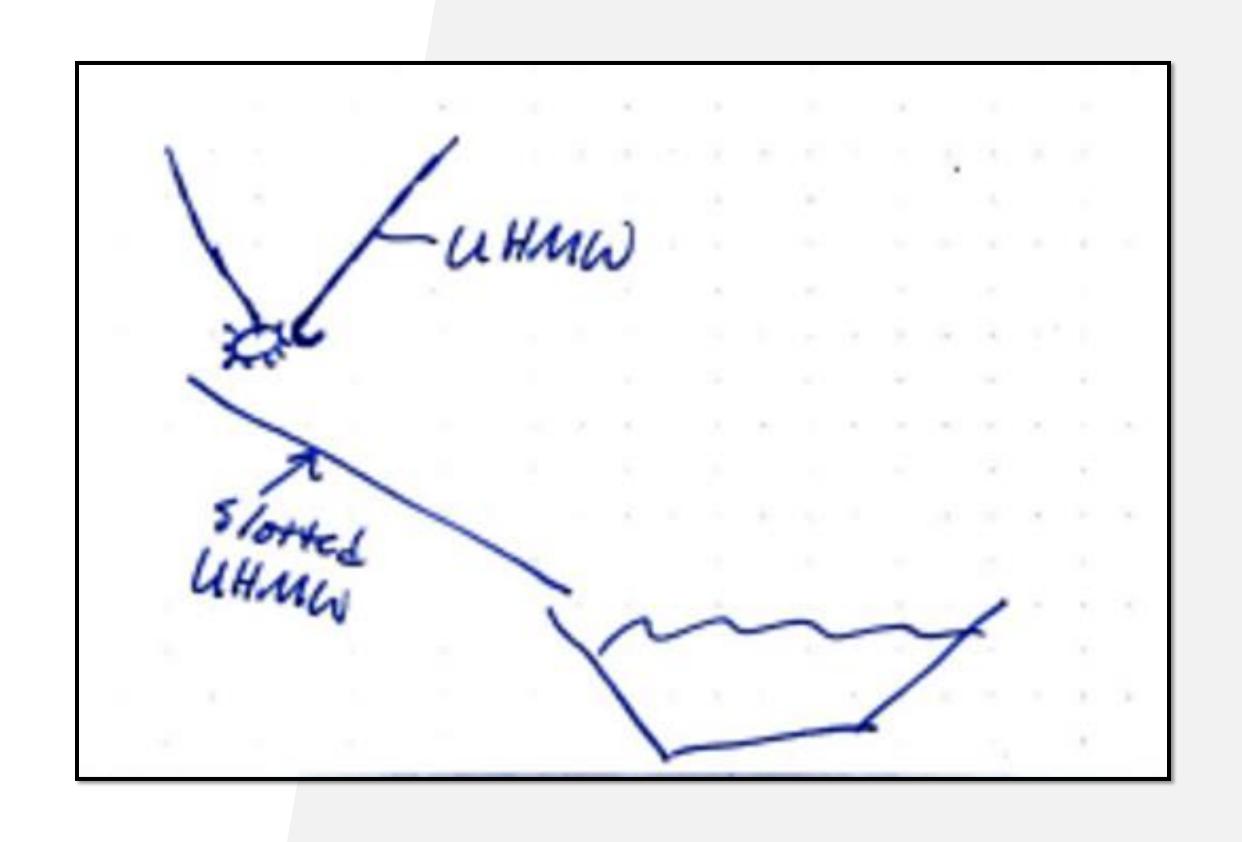


- Fish are poured into top of chute with water
- The slot at the bottom has 1 needle roller and 1 smooth roller that the fish will roll over
- Fish exit the chute, and slide down a slotted ramp into the vaccine



# DESIGN IDEAS IMPLEMENTING USE OF MICRONEEDLE ROLLERS CONT.


## PIPE AND FUNNEL:

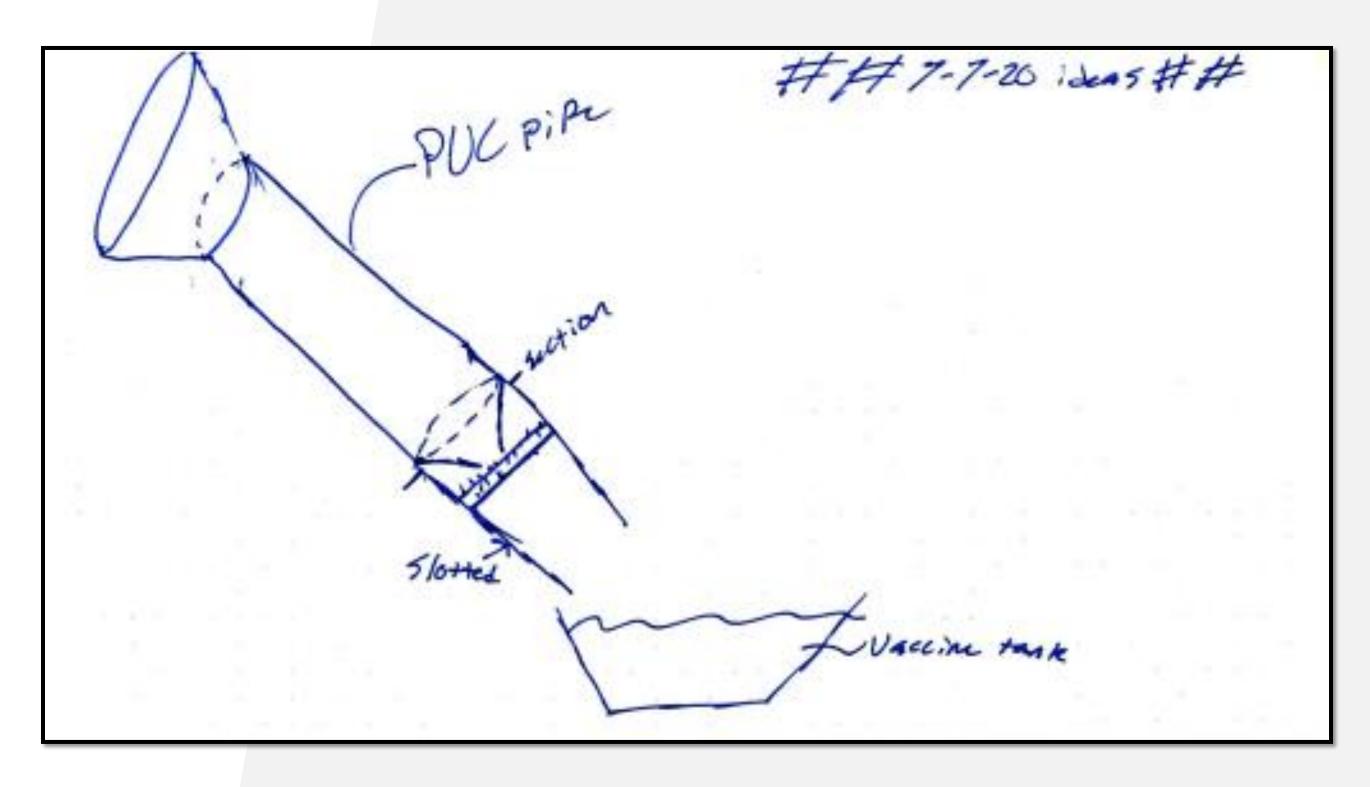

- Fish are poured into the funnel with water
- A cone within the pipe will direct fish into a needled roller located at the exit of the cone
- Pipe below the cone will have perforations that will drain off excess water as fish slide into vaccine


# I

# DESIGN IDEAS IMPLEMENTING USE OF MICRONEEDLE ROLLERS CONT.

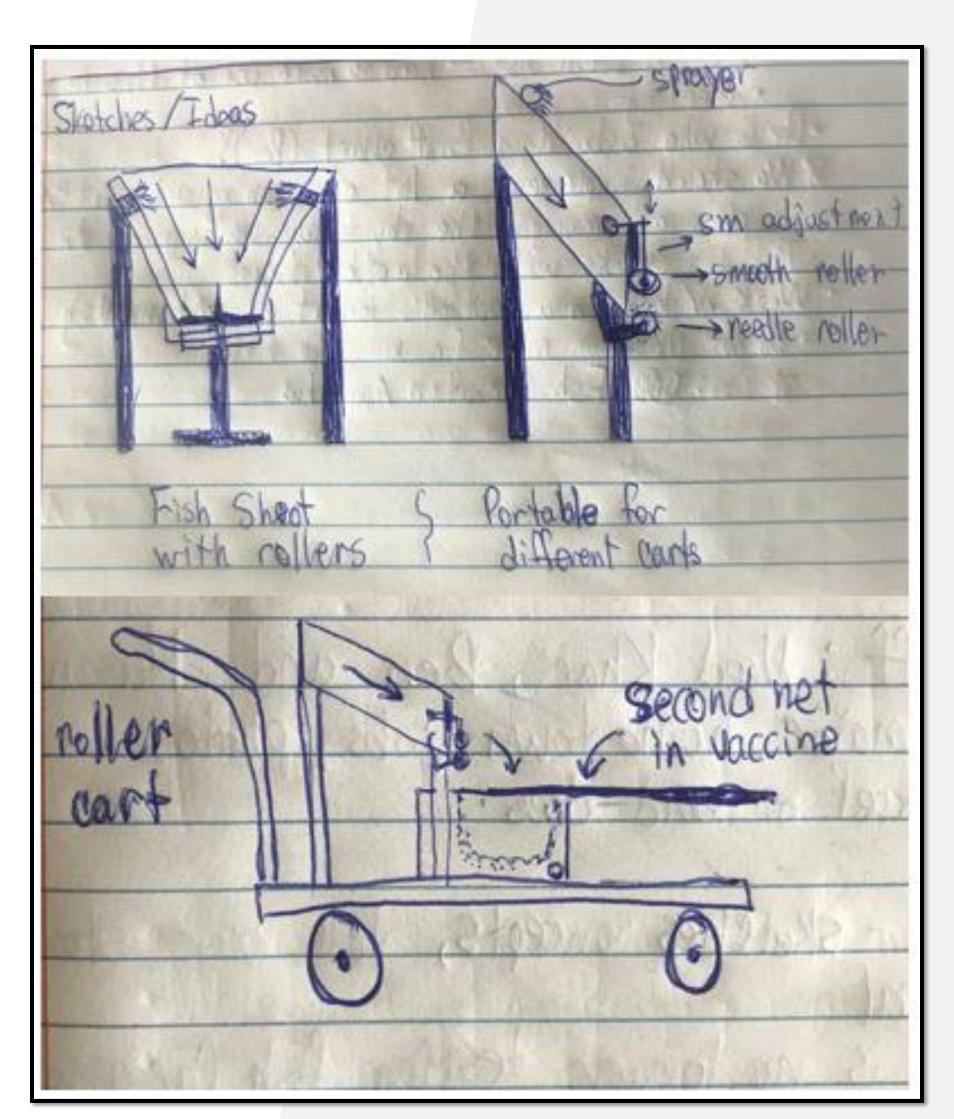

## PORTABLE CHUTE AND CART:

- Fish are poured into the top of the chute
- A small sump pump inside of the tank will supply to sprayers with vaccine solution to assist with fish sliding down the chute
- A stationary needle roller and adjustable smooth roller will be located at the chute exit
- As fish exit, they will fall into a net submerged in the vaccine tank
- The chute will be able to mount to a cart for easy transportation
- Tanks will vary in sizes and will be removable

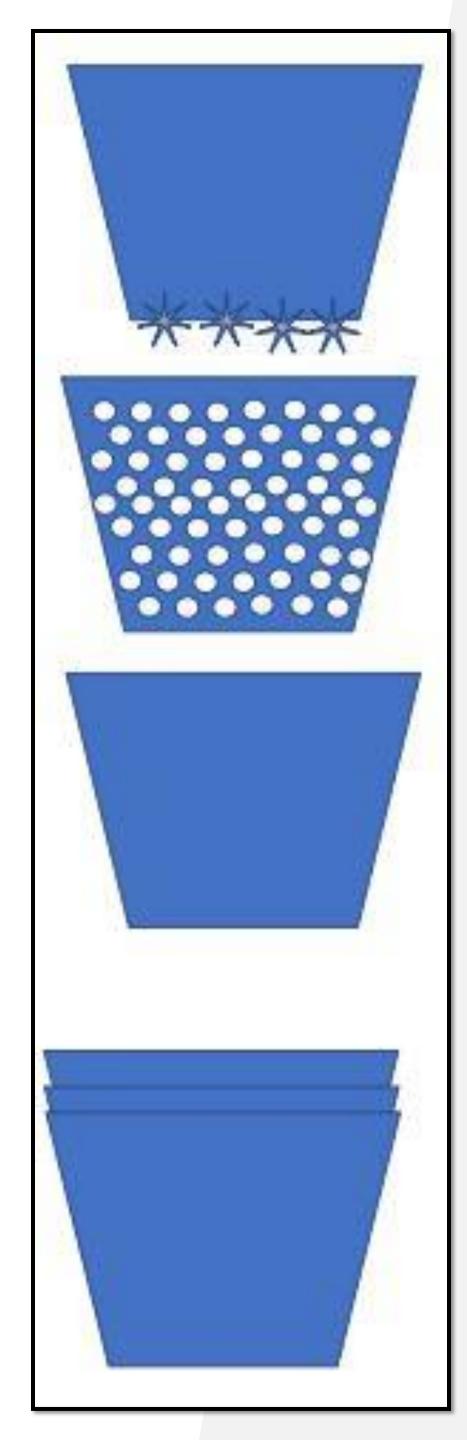

# DESIGN IDEAS IMPLEMENTING USE OF MICRONEEDLE ROLLERS CONT.


## STACKABLE BASKETS:

- Fish are poured into the stacked baskets
- Fish fall though the top basket and pass a series of needled rollers as the basket is lifted
- Middle basket is perforated and allows the fish to be removed from the vaccine tub without draining the vaccine

# DESIGN IDEAS IMPLEMENTING USE OF MICRONEEDLE ROLLERS CONT.




### ABRASION PLATE AND PERFORATED BOWL:

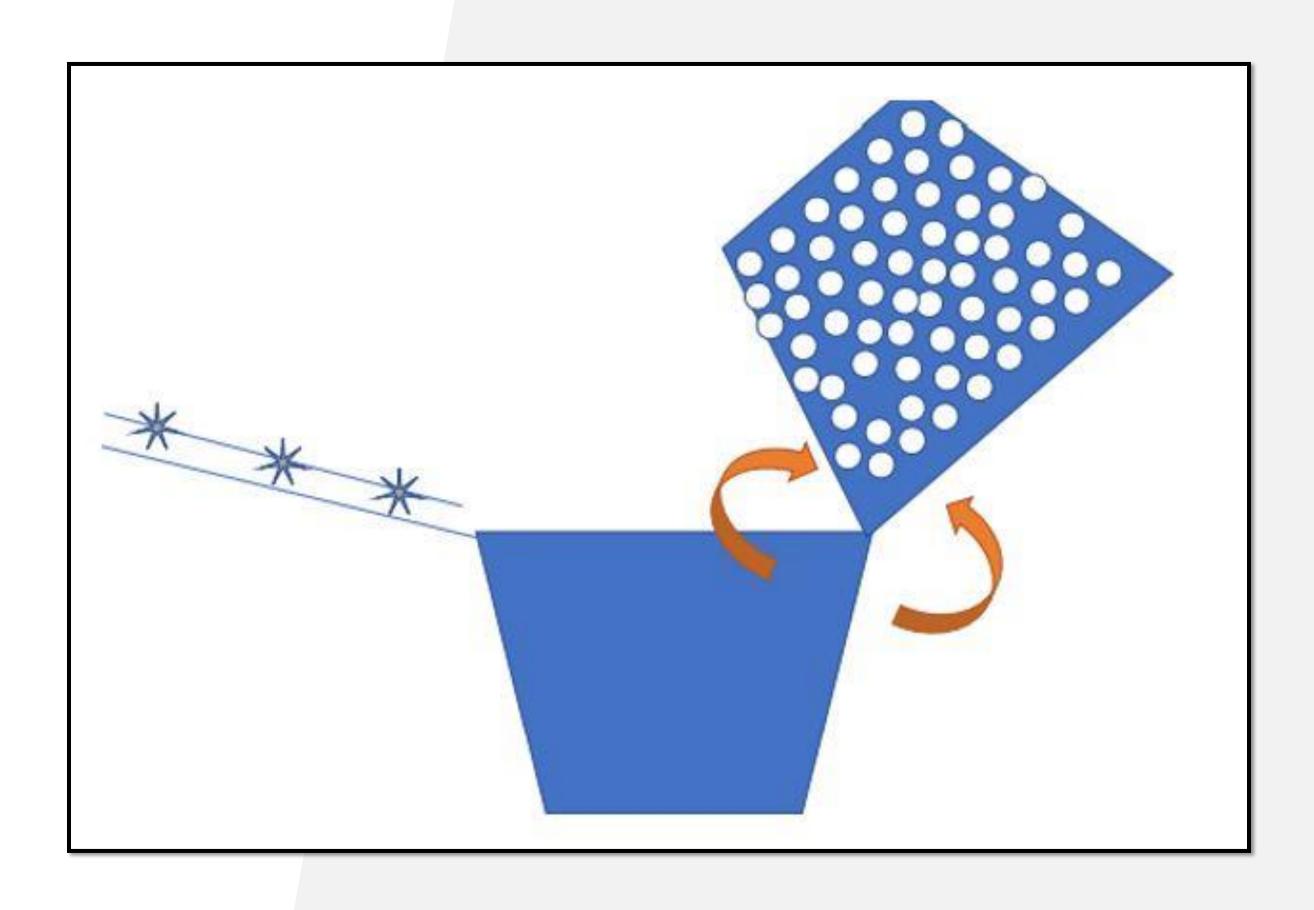

- Fish are poured onto abrasion plate where they are punctured by rollers
- Overflow plate catches the fish, and they slide into the vaccine bowl
- A perforated bowl inside of the vaccine bowl is attached with a hinge that allows fish to be poured out



# LAB TEST MODEL

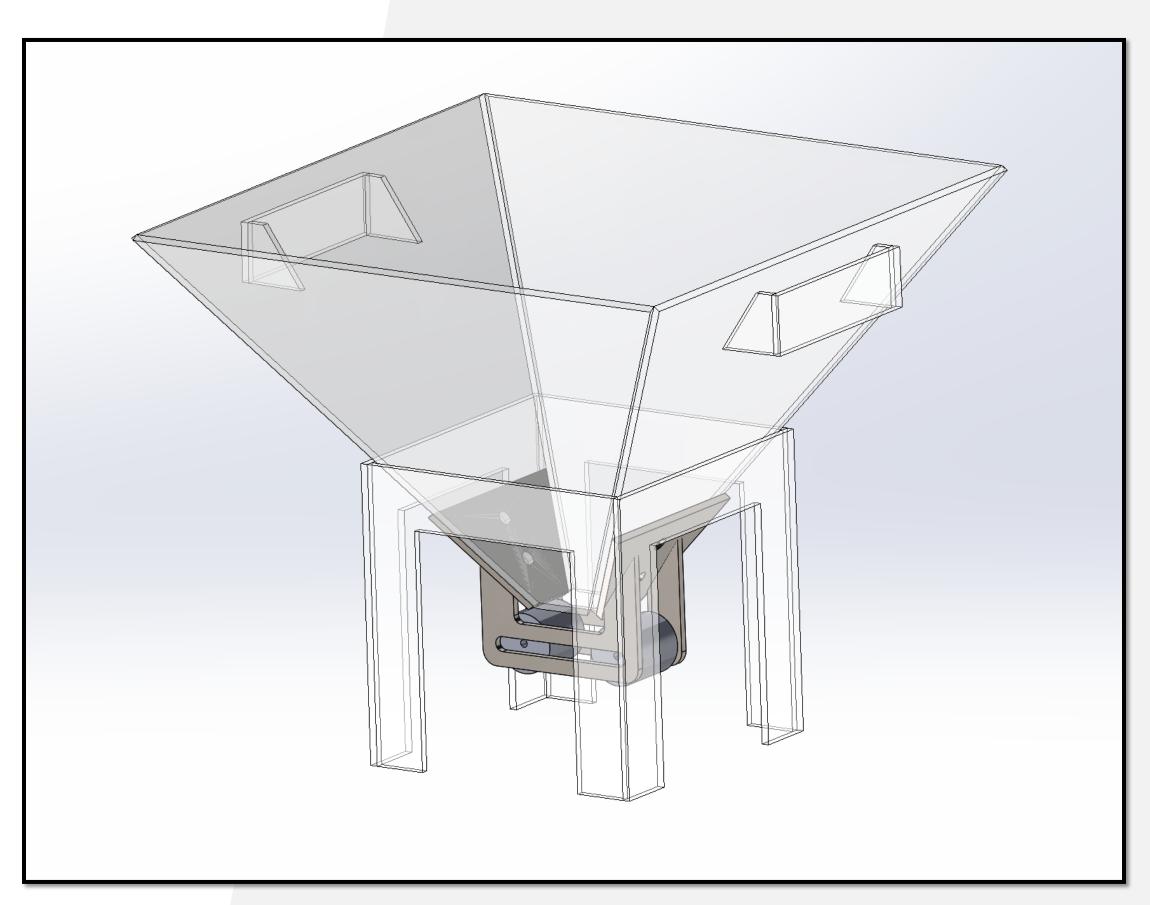

- Stand and Funnel Material: Acrylic
- Roller Mounts: Stainless Steel
- Weight of Funnel with Rollers: 1 lb. 8 oz.
- Top Opening Dimension: (10 x 10) in.
- Bottom Opening Dimension: (1 x 1) in.
- Stand Height: 5 in.
- **Stand Width:** (5 x 5) in.
- Distance of Rollers from Bottom of Tank: 1.25 in.



# DESIGN VALIDATION

- Fish carcasses : 7/24/2020
- Lab Test prototype: 8/7/2020
- Fish carcasses : 9/18/2020
- •Initial prototype: 9/18/2020
- •Final prototype: 11/13/2020

# LAB PROTOTYPE VALIDATION


Before we can start work on our final prototype,, we have a few steps first...

- Institutional Animal Care and Use Committee (IACUC)
- Proof of design and concept effectiveness

Once this is completed, we will be able to start testing and move forward with our final design.



# FISH SKIN PUNCTURE EXPERIMENT

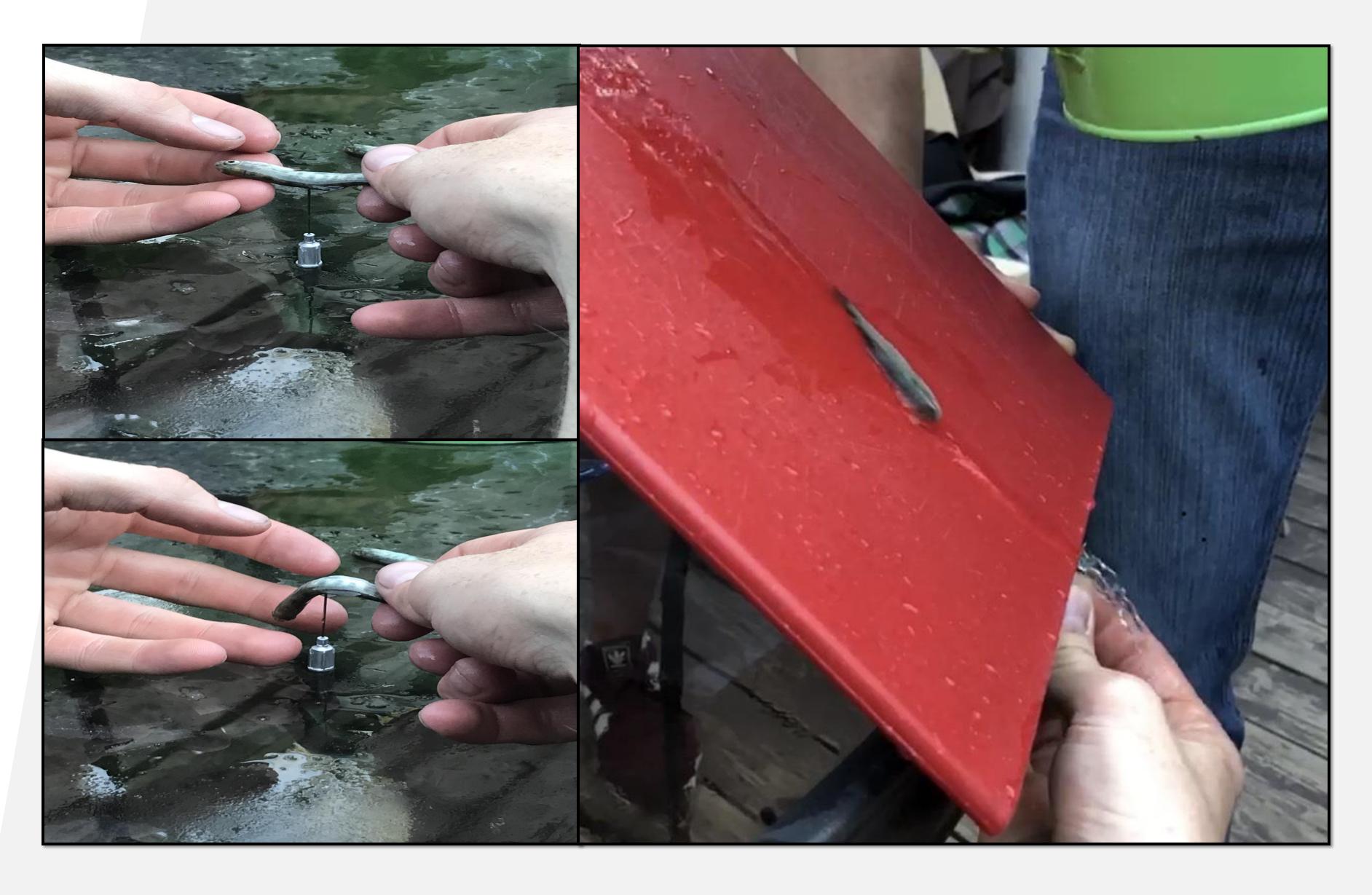



# PROJECT MANAGEMENT

## BUDGET

Capstone Project Budg	get ·	- Sum	me	er/Fall	l									
Project:														
Last Updated														
		2020												
Expense Items	J	July		Aug	S	ept		Oct	Nov	Dec		Total		
Supplies / Parts / Services:		<del>                                     </del>												
Acrylic Sheet			\$	40							\$	40		
Bonding Agent			\$	10							\$	10		
Roller Mount			\$	20							\$	20		
Final Material							\$	500			\$	500		
Cutting/Shaping									\$ 200		\$	200		
Needles/roller			\$	100			\$	100			\$	200		
Stand material							\$	200			\$	200		
Fasteners					\$	50					\$	50		
Pump/Hoses							\$	200			\$	200		
Rapid Prototyping	\$	20									\$	20		
											\$	-		<b>Expenses</b>
													\$ 1,440	Subtotal
			University O				Overhead (		only for SE	A projects)	\$ 95.24		5% of all e	xpenses
										Totals	\$	1,535	\$ 2,000	\$ 465
													Budget	Excess



# PROJECT MANAGEMENT

# SCHEDULE

Fish-Vac Project Schedule													Comp	lete												
		Project:	Altern	ate Aq	uacultu	ire Vaco	cine De	livery S	ystem				In Pro													
		Project: Alternate Aquaculture Vaccine Delivery System  Last Updated: 7/2/2020									Future	_														
		Year																								
	1	Month				July				Aı	ug	ıg			Sept			Oct			Nov					
Milestones Task Description	Duration	Assign To		6/22	6/29	7/6	7/13		7/27	8/3			8/24	8/31			9/21	9/28	10/5 10/1		9 10/26	5 11/2	11/9		11/23	11/30 12/
Snapshot 1									-																	
7/13/2020 Define the requirements	2 Weeks	Ben		0%	100%																					
Brainstorm concepts	3 Weeks	Team			0%	50%	100%																			
Initial Plan for Development	3 Weeks	Cody				0%	50%	100%																		
Snapshot 2																										
8/6/2020 Design Validation	3 Weeks	Ben/Cody					0%	50%	100%																	
Needle Testing	2 Weeks	Team					0%	100%																		
Initial Design Proposal	1 Weeks	Team						100%																		
Value Proposition	2 Weeks	Khari							0%	100%																
Lab Test Prototype	5 Weeks	Ben						0%	25%	50%	75%	100%														
Lab Testing	6 Weeks	Ben									0%	20%	40%	60%	80%	100%										
7/27/2020 Concept Design Review (Bench Test Model)	2 Weeks	Cody						0%	100%																	
Snapshot 3																										
10/15/2020 Engineering Release (Drawings)	6 Weeks	Hasan										0%	20%	40%	60%	80%	100%									
Prototype test results	1 Weeks	Team															100%									
Final prototype design validation	1 Weeks	Cody															100%									
Snapshot 4																										
12/6/2020 Final Protoype	11 Weeks	Ben													0%	11%	22%	33%	44% 559	% 669	6 77%	88%	100%			
Validation and Testing	4 Weeks	Team																						0%	33%	66% 100%
Final Portfolio	3 Weeks	Cody	Ī																						0%	50% 1009



# I

## FINAL PROTOTYPE



- Increased size to handle more fish
- Implements a pump and nozzles to help move fish through the chute
- Can be carried by 1 person
- Adjustable frame for a range of tank sizes
- Chute separates from frame to make the system easier to transport

