Smooth Sanding Device For Arthroscopic Surgeries Jett Murray, Maclean Landis, Lucas Dibelius, Ryan Crowell

Objective

The goal of this project is to create a reliable and safe arthroscopic sanding device that utilizes a random orbital motion with reusable and interchangeable tooltips that shave rough arthritic joint surfaces smooth to reduce need for total knee replacement, lessen pain and lower post-operation rehabilitation time.

Background

- Osteoarthritis (OA) is a characteristic roughening of the joint, causing chronic pain and inflammation
- ~750k knee arthroscopies occur annually in US for OA pain mitigation, but costs are high due to single-use tooltips

Commonly used arthroscopic shaver heads (shown above) often leave grooves in the areas that have been shaved. This causes irritation of the joint tissues and leads to a longer period of rehabilitation.

Value Proposition

- Currently, no technology exists to re-shape damaged joint surfaces, which could lower patient rehabilitation times
- This device aims to create a random-orbital sanding platform for OA arthroscopic surgeries to improve patient outcomes
- Allow reshaping of bone and cartilage to treat OA pain
- Incorporate reusable tooltips to reduce patient costs

Key Requirements

- Variable output speed from 900-6000 RPM
- Random orbital sanding hypoallergenic tooltips
- Corrosion-resistant tooltip/output shaft of 4mm diameter
- Compatibility with modular tooltips
- Different grits
- Reliable 2-hour battery life with modular power supply
- Compatibility with existing surgical suction devices
- Debris (bone/cartilage) removal
- Reusable device with reusable tooltips capable of sterilization
- Autoclavable

Concept Development

- Two initial designs: Sawzall DC motor and linear actuator, fixed DC motor with linear solenoids
- Reliability concerns directed to second option
- Completed circuit and coding
- Inner mechanism CAD done
- 3D Printed scaled models
- Handle test bed for DC motor
- CAD handle in progress
- Awaiting purchased parts to begin prototyping with solenoids on test bed

The Final Design

Design functions with DC motor mounted linearly to the rotating shaft, conjunctive to 4 micro solenoids working in tandem to create linear actuation

mating they Vantell Here and to output volary motion, her pour source TDD - liventy + Power supply in handly

- Achieves linear and rotational motion
 - Potentiometer controls revolutions, toggle to activate
 - Cylindrical rounded shaving head, sand blasted
 - Prototype #1
 - 3D printed handle and inner orbital sanding mechanism
 - Stock metal outer rod, inner rod 3D printed
 - Potentiometer and toggle activation
 - Enclosed battery and circuitry
 - Limited head sanding potential
- Final Prototype
 - Machined handle and inner orbital sanding mechanism
 - Handle from stainless steel, inner rod and head from stainless steel, outer rod from aluminum with noncorrosive coating
 - Sandblast head, full sanding capability



Validation

To determine the functionality of our early-stage prototype we have put the machine through a few tests:

- Solenoid strength testing results: repeating four different times we reached an average pulling strength of 255 grams
- Shaft actuation testing results: repeating this several times we reached a conclusion that the shaft needed 100 grams in line with shaft
- We went through various CAD models to ensure that the model was best suited for surgical use. Considerations included comfortability, weight balance, and visibility







- Actuation fork for the shaft creates a moment that can cause the motor to fail
- Spring does not have required force to pull fork back
- Upcoming tests: battery longevity, contact testing, comfortability quality testing,

Summary, Conclusions, and Recommendations

Solenoid concerns lead us to using four solenoids using twosided actuation fork (in front and behind fork both above and below it)

With motors up and running and CAD models near completion next steps will be:

- Designing and testing shaver heads
- Machining CAD model
- Machining shaft actuation parts
- Ensuring autoclavibility
- Compare performance of shaver with commonly used shavers
- Performing final tests to ensure Shaft stays perfectly steady in use

Acknowledgements



The team would like to thank the client, Dr. Doug Hiller, for his support and contribution to the project.

